



MoliCare® (premium) Fixpants & Lindor® Netpants

EN Instruction for processing of medical devices

Product

MoliCare® (premium) Fixpants
Lindor® Netpants

Product description

Elastic fixation pants with long legs/ short legs.

Intended use

MoliCare® Premium Fixpants and MoliCare® Fixpants and Lindor® Netpants are non-invasive, non-sterile, reusable fixation systems for safe and hygienic securing of all types of incontinence pads for long term contact with the body used by lay persons and/or health care professionals for humans on intact skin.

Processing limitations

 MoliCare® Fixpants short leg: The supplied product is non-sterile and washable up to 10 times at 60 °C.

Tumbled dry at low heat. Do not bleach, do not iron and do not dry clean.

MoliCare® Fixpants long leg & Lindor® Netpants: The supplied product is non-sterile and washable up to 25 times at 60 °C. Tumbled dry at low heat. Do not bleach, do not iron and do not dry clean.

MoliCare® premium Fixpants short leg: The supplied product is non-sterile and washable up to 25 times at 60 °C. Tumbled dry at low heat. Do not bleach, do not iron and do not dry clean.

MoliCare® premium Fixpants long: The supplied product is non-sterile and washable up to 50 times at 60 °C. Tumbled dry at low heat. Do not bleach, do not iron and do not dry clean.

Special precautions

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Product disposal

To minimize the risk of potential infection hazards or environmental pollution, disposable components of HARTMANN should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Cleaning Instructions

Storage and transportation	Storage condition: "dry, indoor only, keep away from open fire and other sources of ignition, protect from direct UV rays, avoid from other chemicals, no permanent uncovered storage" Storage temperature: "unclimated storage conditions" Storage time: "storage stability 5 years"
Preparation at place before cleaning/ disinfecting	There are no special preparations needed before using the fixation pants. Only unpacking the fixation pant from the bag.
Packaging	Fixation pants are packed in PE-bag for transportation. No special packaging systems needed.
Degree of degradation	Depending on storage time and washing cycles (mentioned before)
Cleaning/Disinfection	No disinfection needed. Cleaning according washing and drying instruction. Washing and drying at maximum 60 °C.
Additional information	Do not bleach, dry clean or iron the product.
Drying Process	Cleaning according washing and drying instruction. Washing and drying at maximum 60 °C.
Inspection, maintenance, testing and packaging	Not applicable

The instructions provided above have been validated by the manufacturer of the Device as being capable of preparing a Device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

Date of revision of the text: 2025-06-30

MoliCare® Fixpants short leg & long leg, MoliCare® premium Fixpants short leg, MoliCare® premium Fixpants long leg, Lindor® Netpants:



(200825)

 PAUL HARTMANN AG
Paul-Hartmann-Straße 12
89522 HEIDENHEIM, GERMANY
www.hartmann.info

