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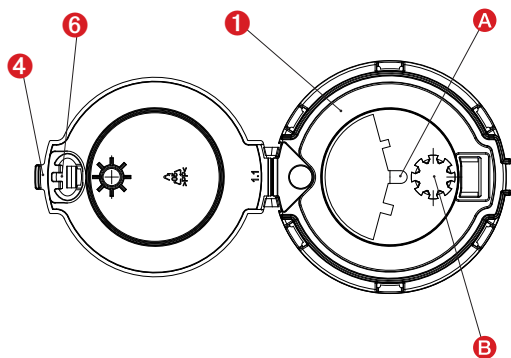
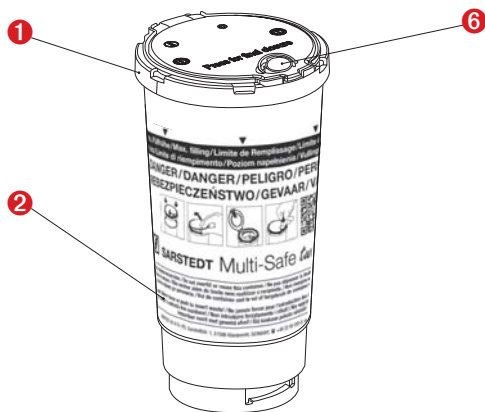
1. Intended purpose and normal use

The disposal box is exclusively intended for the disposal of pointed and sharp items like, for example, needles and scalpel blades for medical use.

2. Safety instructions

- Use only boxes adequately sized to accommodate the items to be disposed.
- Keep the box beyond the reach of children.
- Ensure that the lid (1) is assembled and locked into place before filling.
- Attach the box to recommended base adapter or wall holder, or place into the Safety-Tray to protect it against overturning.
- Ensure that items are placed into the box in a vertical direction only.
- Scalpels must be placed into the box with utmost care. Using too much force or filling other objects on top involves the risk of jamming and damage to the box walls or base.
- Do not use force to press objects into the box.
- Do not fill with liquids.
- Discard insulin pens through the opening.
- Do not fill the box above the maximum filling line (5) (health hazard!).
- Do not throw. Do not shake. Do not drop. Do not shake.
- Ensure that no objects are jutting out before closing.
- Do not press the box against your body (e.g. chest) for final closure.
- Do not press the box together from the outside when full.
- Prior to disposal of the box, ensure that lids (1) and day lid (3) are securely closed.

3. Assembly (lid assembly)

Prior to filling, place the box on a solid surface and press the lid (1) firmly onto the box base (2).

The lid (1) audibly locks into place along its entire circumference.

If pressing the entire lid once is too forceful, you can start pressing at one point until it locks into place and continue step by step.

4. Daily use

For daily use, press the day lid (3) into the filling opening until it audibly clicks into place. By pushing up the flap (4) a reopening is possible.

5. Final closure of day lid (3)

After filling [Warning: Do not fill above the maximum fill line (5)!], proceed as follows: Place the box on a solid surface, press the day lid (3) into the filling opening until it audibly clicks into place and press the little Button (6) "Press for final closure". (The box is finally closed).

6. Fill line - maximum filling level (5)

The upper section of the label features a fill line (5). Ensure that the box is not filled above this line (5). For safety reasons, do not fill the box above the maximum filling level (5).

7. Holding fixtures

A base adapter, wall holder or Safety-Tray is available to protect the box against unintentional overturning (Refer to separate User Guide: "Holding Fixtures").

8. Removal aids

8.1 U-shaped recess (A)

For sharps removal from Luer lock systems.

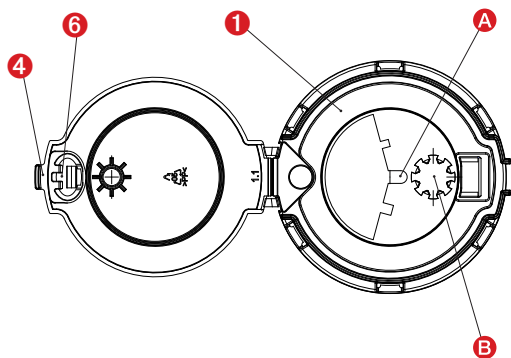
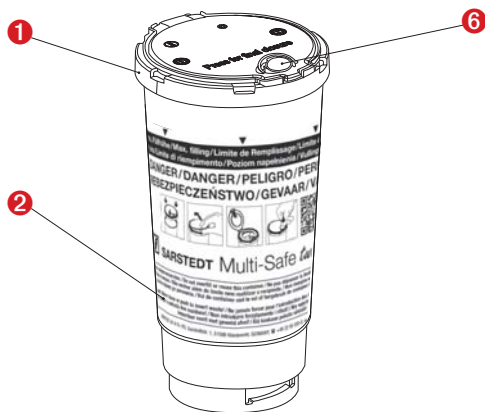
8.2 Star-shaped, notched opening (B)

For the disassembly of insulin pens.

9. Disposal and identification

Disposal boxes are made from ecologically friendly polypropylene. Incineration does not cause emission of any harmful substances. Observe statutory provisions pertaining to disposal! Technical modifications reserved.

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